


Medical Device Alert

Action

Ref: MDA/2009/057 Issued: 20 August 2009 at 10:00

Device

Smart mobile hoists manufactured by Molift Group AS.

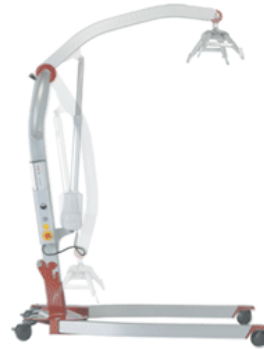

Problem

Potential for the collapse of the hoist mast when in use due to incorrect assembly. This can result in serious injury or death.

Action by

All those involved in the provision, maintenance or use of these hoists. In particular: nurses, care staff, back care/manual handling advisors and maintenance staff/contractors.

CAS deadlines

Action underway: 03 September 2009
Action complete: 20 November 2009

Action

1. Trace all Smart mobile hoists manufactured prior to week 11 of 2009.
2. Obtain new safety labels and copies of the revised instructions for use from the UK supplier.
3. Arrange for the new safety labels to be fitted.
4. Ensure all users are aware of the revised assembly instructions.

Contact

Supplier

Kenneth Finnan
Meditec Molift Ltd

Tel: 0844 800 4236

Fax: 0844 800 4237

E-mail: Ken.Finnan@meditecmedical.com

[Link to full Medical Device Alert](http://cmsupgradepd/Publications/Safetywarnings/MedicalDeviceAlerts/CON054622)

<http://cmsupgradepd/Publications/Safetywarnings/MedicalDeviceAlerts/CON054622>