

APPENDIX A

Ferno (UK) Ltd, Ferno House,
Stubs Lane
Cleckheaton
West Yorkshire BD19 4TZ
Tel: 01274 851999
Fax: 01274 851111

Ferno (UK) Ltd

Fax

To: From: Bob Williamson - Quality
Manager
Fax: Pages: 1
Date:
Re: Compact Carry Chairs CC: N/A

MAINTENANCE ADVICE FOR FERNO CARRY CHAIRS
PLEASE ENSURE THAT YOUR MAINTENANCE DEPARTMENT ARE
ADVISED AS TO THE FOLLOWING ADDITION TO THEIR USER MANUAL
FOR CARRY CHAIRS.

ESSENTIAL REQUIREMENT

Every three months lubricate the hinge pins with light oil, paying particular attention to the cross section pins.

Although there is already a recommendation within the user manual to lubricate the chairs it is felt that a lack of lubrication could aggravate deterioration of the pivot points and lead to a potential failure of the pivot rivet, hence the addition of this essential requirement.

For information, the rivets on future chairs will be replaced with high tensile steel bolts to aid maintenance and cover replacement.

Should you have any questions related to this notice then please contact the undersigned or the Ferno (UK) Ltd sales team.

Should you require any service or maintenance to be conducted on your carry chairs then please contact Tricia Cowler in Fieldcare on 01274 854511.

Yours sincerely



Bob Williamson
Quality Manager