



Technical Information

Bulletin No: T1004
Date: 28 September 2001

SAFETY BULLETIN - POSSIBILITY OF LOOSE SCREWS ON POWER CHAIR DRIVE WHEELS

Due to the insufficient application of a locking compound on the rear drive wheels on some models of our powered wheelchairs, we have found that it is possible for the centre screw holding the rear wheel to the drive shaft to come loose.

This only affects **Cruiser, Harrier HD, Harrier Plus, Harrier Special, Home and Away, Home and Away Special, Phoenix Plus, Spectra Plus and Spectra Special** powered wheelchairs made between **1st June 2001** and **26th July 2001**.

From our records you have been supplied with some of these wheelchairs and would have provided them to your customers.

To eliminate any possibility of the rear wheels coming loose, please carry out the rectification action by removing the central fixing screws from both rear wheels and replacing them with the screws provided. These have locking compound already applied to the screw threads.

Invacare Limited will reimburse you the sum of £35.00 for each chair you rectify.



**Replace this central fixing screw
on both rear drive wheels**

To advise us that both the screws have been replaced, please complete the information in the attached list and return it to:

Mr Kevin Humphries. Director Technical Services
Invacare Ltd, South Road, Bridgend Industrial Estate, Bridgend, Mid Glamorgan,
CF31 3PY.